PIPETTE / SERINGUE DOSEUSE DOS'UP[®], DOS'UP[®] PLUS

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DOS'UP[®], DOS'UP[®] PLUS
MEASURING PIPETTE/SYRINGE
Dosage for pediatric syrups and oral suspensions
INSTRUCTIONS FOR USE

DESCRIPTION

DOS'UP[®] Pipettes/Syringes are used to dose and dispense pediatric syrups and oral suspensions. They are made of two parts: a barrel and a graduated plunger.

The technology to manufacture DOS'UP[®] Pipettes/Syringes is molding plastic injection. The raw material is food contact and European and United States pharmacopeias compliant. Some references contain a food contact compliant colorant.

DOS'UP[®] Pipettes/Syringes are for individual use. The patient must follow the instructions given by the pharmaceutical industry on the package leaflet. Please take the time to read these operating instructions carefully before using the device. They may be used once or several times uses for a single patient. The device provided should be used with the particular drug with which it is included.

USE

Before the first use, it is necessary to watch manually the pipette/syringe with water.

Two different applications: with or without adaptor.

1. A) With adaptor

• Introduction of the adaptor :

- Check that the pipette/syringe is not damaged and well cleaned,
- Open the vial,
- Insert the adapter into the neck of the bottle until abutting,
- Insert the pipette/syringe into the adapter until it stops.

• Dosing:

- Return the vial (bottle neck in direction of the floor),
- When dosing, it is recommended to firmly hold the vial and the body of the pipette/syringe,
- Pull slowly the plunger to withdraw the drug, to the line corresponding to the dosage indicated on the leaflet,
- Put the bottle back in its original position and remove the pipette or the dosing syringe from the adapter.

NB: The adaptor is an accessory to facilitate dosing, but has not been qualified as a primary packaging by UNION PLASTIC.

B) Without adaptor

• Dosing:

- Check that the pipette/syringe is not damaged and well cleaned,
- Open the vial,
- Immerse the pipette/syringe into the medicine by having verified that the piston is pushed well,
- Pull the plunger to withdraw the drug, to the line corresponding to the dosage indicated on the leaflet.

2. Application:

- Release the contents of the pipette/syringe into the child's mouth as described in the leaflet by applying a regular and slight thrust on the piston.

3. Cleaning after use:

- Separate both parts of the pipette/syringe (barrel and plunger),
- Wash them carefully by hand with water until disappearance of medicinal residues,
- Allow to air dry before,
- Reassembly the parts when completely dry,
- Store the pipette/syringe with the syrups in its box in a place inaccessible to children.

INDICATIONS FOR USE

DOS'UP[®] Plus Pipettes/Syringes have to be used depending on the customer's modus operandi and on the dispensed product.

Any serious incident involving the device should be notified to the manufacturer and to the competent authority of the Member State in which the user and / or patient is established.

CONTRAINDICATIONS

- Do not use the pipette/syringe if it is damaged or dirty.
- Do not use the pipette/syringe for any other application or with any other medication.
- Do not use for administration in other ports.
- The pipette/syringe cannot be used beyond the expiration date of the medication.

DEGREE OF MEDICAL DEVICE ACCURACY

The dosing accuracy is 5% for pipette/syringe <1ml and 10% for the pipettes and syringes > 1ml.

PRECAUTIONS OF USE

DO NOT USE THE DEVICE IF IT'S DAMAGED.

The washing temperature must not exceed 50°C: do not wash in the dishwasher, do not boil.

The products should be used by people who have no insanity of mind or impaired mental faculties, and can read and understand the instructions.

Keep out of reach of children.

Check that the package is not damaged.

CONDITIONS OF STORAGE

The pipette/syringe should be kept clean and dry in the medicine's packaging at a temperature between 5°C and 50°C.

The pipette/syringe must be disposed of at the same time as the container of the drug with household waste, or preferentially return to the pharmacist all the opened boxes as well as the vials for an appropriate destruction.

MANUFACTURER INFORMATION

First placing on the market: 1998

Date of last revision of the instructions for use: Notice DT3 revision D 2020/10/29

Number of the notified body for the CE marking identification: 0459

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