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VAGINAL APPLICATOR INSTRUCTIONS FOR USE

DESCRIPTION

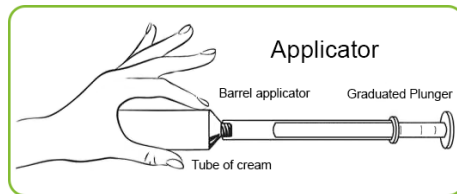
Vaginal and gynecological applicators are used to dose and dispense preparations as medicated or cosmetic paste or cream. They are made of two parts: a barrel and a plunger and sometimes a cap. The technology to manufacture vaginal and gynecological applicators is molding plastic injection. The raw material is food contact and pharmacopeia compliant (low-density polyethylene and/or polystyrene). Some references contain a colorant; one or several adjuvants (slipping agent). They may be packed loose or in sealed plastic bags (containing 1, 5 or 10 syringes). The purpose of this kind of packaging is not to maintain the sterility or the cleaning. Vaginal and gynecological applicators are **used by only one person**. The patient must follow the instructions given by the pharmaceutical industry on the package leaflet. They may be used once or several times. The device provided should be used with the particular drug with which it is included.

USE

Before the first use, it is necessary to wash manually the applicator with water.

Before each use, slightly press a piece of cream out of the tube and pull the plunger of 2 to 3mm before screwing the applicator. It will help to reduce the air bubbles and to get the required volume of product.

- **Wash hands before use**
- **Dosing:**
 - Remove cap from tube and screw the open end of the applicator onto the tube,
 - Gently squeeze the tube, pushing the medication into the open barrel or to the line corresponding to indicated dosage,
 - The applicator contains the recommended amount when the plunger stops.

- **Application:**
 - Unscrew the applicator and apply the product as described in the leaflet.
- **Cleaning:**
 - Separate both parts of the applicator (barrel and plunger),
 - Wash carefully them with warm, soapy water,
 - Rinse thoroughly with warm water,
 - Allow to air dry before,
 - Reassembly the parts when completely dry,
 - Store the applicator with its syringes in its box in a place inaccessible to children,
 - The syringe will be used according to the prescription and no longer than the duration of the treatment.

INDICATIONS FOR USE

The applicator should be used according to the procedure of the pharmaceutical manufacturer and according to the drug to be administrated.

This medical device is reusable until the end of treatment.

The instructions for use should be translated into the official languages of the country of use in the package leaflet.

Any serious incident involving the device should be notified to the manufacturer and to the competent authority of the Member State in which the user and / or patient is established.

CONTRAINDICATIONS

- Do not use the applicator if it has been badly cleaned or damaged.
- Do not use the applicator for another application or for any medicine other than the one used
- Do not use for administration in other ports.
- The applicator cannot be used beyond the expiry date of the drug.

DEGREE OF MEDICAL DEVICE ACCURACY

The general accuracy of the applicator is $\pm 20\%$ for all volumes.

PRECAUTIONS OF USE

DO NOT USE THE DEVICE IF IT IS DAMAGED.

The medical device is delivered in the same packaging as the medication.

The washing temperature must not exceed 50 ° C: do not wash in the dishwasher, do not boil.

The product should be used by people who have no insanity of mind or impaired mental faculties and can read and understand the instructions.

Keep out of children.

Check that the packaging is not damaged.

CONDITIONS OF STORAGE

The applicator should be kept clean and dry in the medicine's packaging at a temperature between 5°C and 50°C.

The cup must be disposed of at the same time as the container of the drug with household waste, or return to the pharmacist all the opened boxes including the cup as well as the vials for an appropriate destruction.

MANUFACTURER INFORMATION

First placing on the market: 1998

Date of last revision of the instructions for use: Notice DT4 revision E 2020/10/29

Number of the notified body for the CE marking identification: 0459

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