VIAL ADAPTERS

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DESCRIPTION

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INSTRUCTIONS FOR USE

DESCRIPTION

These instructions for use must be read before using the Vial Adapter.

Vial Adapter is a single used medical device indicated for the transfer and mixing of drugs contained in vials.

Vial Adapter is sterilized by Gamma Irradiation.

Vial Adapter is individually packaged in a rigid solid sealed blister.

Vial Adapter must be used with the drug for which it is prescribed and according to the pharmaceutical manufacturer's instructions for use.


Vial Adapter is compatible with NaCl-based solvents. It is passive and has no interaction.

Compatibility of Vial Adapter with the drug is the responsibility of drug's manufacturer.

Vial Adapter is manufactured by UNION PLASTIC by injection of plastic materials compliant with European and United States Pharmacopias.

USE

1. Before handling the components, wash your hands with soap and hot water. Put-on medical gloves. Use it in clean and healthy environment.
2. Remove the protective cap from the vial. Clean aseptically the vial rubber stopper with alcohol, be careful not to handle the rubber stopper directly with hands.
3. Place the vial on a clean and flat surface. Peel off the paper lid of the blister. Do not remove the vial adapter from the blister.

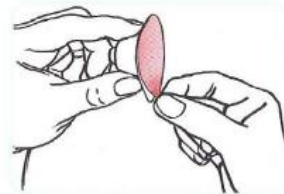4. Hold the blister, place over the vial and firmly press down and snap it. Remove the blister and throw it away.


5. Attach the prefilled syringe to the threaded Vial Adapter by turning clockwise .


6. Transfer the diluent to the vial by slowly pushing the syringe plunger all the way in.


7. Gently swirl the vial to make sure the product is thoroughly mixed. Do not shake vial. Be sure that the powder is completely dissolved.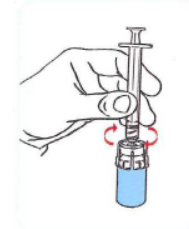

8. Withdraw solution into the syringe by holding and returning the vial on the Vial Adapter and syringe, then draw the plunger rod out slowly and smoothly. Ensure that the entire content of the vial is drawn into the syringe.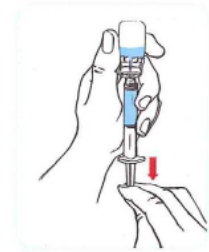

9. With the plunger rod in place, remove the syringe from the vial adapter by gently pulling and turning the syringe counterclockwise (the vial adapter should remain attached to the vial). Throw away in the appropriate way. The mixing is now ready for administration.

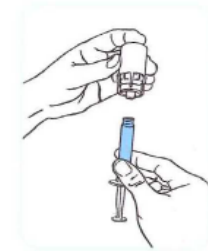VIAL ADAPTERS

INSTRUCTIONS FOR USE

INDICATIONS FOR USE

Vial Adapter must be used according to pharmaceutical manufacturer drug instructions for use.

It is a single use Medical Device.

These instructions for use should be translated in drug's instructions for use in using country official language.

Any serious incident related to the device must be notified to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Vial Adapter is Non-Irritating, Non-cytotoxic and Non-sensitizing to skin and eyes.

CONTRAINDICATIONS

- Do not use Vial Adapter if its packaging is damaged or broken,
- Do not use Vial Adapter if it is damaged,
- Do not use Vial Adapter for another application or with another drug than it is prescribed for,
- Do not use Vial Adapter when expiry date has been exceeded.
- Do not wash, re-sterilized, sanitize, boil or freeze Vial Adapter.
- Do not re-use Vial Adapter.

PRECAUTIONS FOR USE

DO NOT USE VIAL ADAPTER IF IT IS DAMAGED OR EXPIRED.

Vial Adapter is ready to be used.

Vial Adapter must be used by people who have no insanity of mind or impaired mental faculties and can read and understand the instructions for use.

Vial Adapter must only be used by a health professional.

Before use, check if Vial Adapter's blister is not damaged.

Keep out of the reach of children

CONDITIONS OF STORAGE

Store in its original sealed container away from direct sunlight and heat, in dry area.

Store at room temperature 20+/-5°C.

Vial Adapter must be kept in its own origin packaging.

After use, Vial Adapter and its packaging must be thrown away in appropriate way.

MANUFACTURER INFORMATION

Claimed expiry date: 5 years minus 1 month. See specific date on each blister.

First placing date on the market: 2014

Class Is Medical Device.

SYMBOL GLOSSARY

Consult Instructions for use

Catalogue Reference

Caution to the device sterilization

Non-pyrogenic

Sterilized by Irradiation

Sterile Barrier System

Expiry date

Manufacturer

European Conformity Mark

0459

Notified Body number

Do not re-use

Batch Number

Do not use if packaging is damaged

Medical Device

Unique Device Identification

R_x Only

Caution: U.S. federal law restricts this device to sale by or on the order of a physician.

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